**AT51206 KACZKA DO ODDAWANIA MOCZU
AT51207 BASEN SANITARNY
INSTRUKCJA OBSŁUGI
AT51206 URINAL AT51207 BEDPANUSER MANUAL
AT51206 URINAL AT51207 BETTPFANNEBEDIENUNGSANLEITUNG
AT51206 MOČOVÁ LÁHEV AT51207 NOČNÍKNÁVOD K POUŽITÍ
AT51206 MOČOVÁ FLAŠA AT51207 TOALETNÁ MISKANÁVOD NA
POUŽITIE
AT51206 URINOIR AT51207 BASSIN HYGIÉNIQUEMODE D'EMPLOI
AT51206 URINEFLES AT51207 BEDPANGEBRUIKSAANWIJZING
AT51206 ORINAL AT51207 BATEA SANITARIAMANUAL DE
INSTRUCCIONES
AT51206 URINALE AT51207 VASO DA LETTOISTRUZIONI PER L'USO
AT51206 URINFLASKA AT51207 SÄNGBÄCKENBRUKSANVISNING**

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En

Do not use the product before reading and understanding these instructions. If you do not understand the warnings, cautions or instructions, contact a health care professional or your dealer to avoid damage to the product.

CAUTION: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

CAUTION:

It is forbidden to use the product other than in accordance with its intended use.

CAUTION:

Before use, check that all components have been assembled correctly.

CAUTION: The manufacturer is not responsible for damage caused by negligent maintenance, inadequate servicing or resulting from failure to follow the recommendations contained in this manual.

CAUTION:

In the event of a device-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

THIS MARK MEANS MAXIMUM USER WEIGHT!

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product

MAINTENANCE AND CLEANING

The product should be cleaned after each use, with commercially available cleaning agents. Do not use rough objects that may damage the surfaces of the product.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

HOW TO DISPOSE OF THE PRODUCT AFTER WITHDRAWAL FROM SERVICE

After the device is withdrawn from service, the medical device can be disposed of as ordinary municipal waste

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this product is intended (see the section on the purpose of the product of this manual). The product may be purchased by the user independently or on the recommendation of a doctor, therapist or other specialist. Whether purchasing the product on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

AT51206

APPLICATION

The sanitary duck facilitates urination for bedridden patients who cannot use the toilet independently. With its help, urine can be easily and conveniently urinated and then removed. The built-in closure ensures hygienic use.

FEATURES:

- The duck is equipped with an additional tip that allows women to use it

USE AT51206

Before first use, clean the product for hygiene. In addition, check the duck for any sharp edges that may have formed during the manufacturing process. If there are any, they should be removed to prevent cuts. Next, the vessel should be carefully brought closer to the intimate parts of the patient's body and positioned in such a way that the urine can pass inside unhindered without escaping. After urination, carefully push the device away, preventing urine from spilling out, and then close the lid and position the device with the opening upwards.

TECHNICAL PARAMETERS

Capacity: 800 ml

Length: 20 cm

Width: 11 cm

Height: 12cm

Weight: 0.1kg

Maximum user weight: 110kg

AT51207

APPLICATION

The sanitary basin is a device designed to deal with physiological needs for people lying down who are unable to use the toilet. It allows one to take care of physiological needs in a lying or semi-sitting position.

FEATURES:

- The pool is equipped with a lid and a handle, which affects the comfort of the product.

USE

Before the first use, clean the product for hygiene. In addition, check the duck for sharp edges that may have been formed during the manufacturing process. If there are any, they should be removed to prevent cuts. If the patient shows any mobility, it is best for him to bend his legs at the knees and raise his hips slightly. If the patient cannot move, he should be gently moved to the left side. The next step is

to slide the bassinet under the patient's buttocks. In the case of a patient who is unable to move on his own - once the bassinet has been placed, the patient should be slowly laid on his back. In order to make the patient more comfortable, a pillow or a rolled up towel/blanket can be squatted over the patient's lumbar region. After the patient is done, slide out the sanitary pool, close the lid and then empty and clean the container as soon as possible.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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