AT51208

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Thank you for purchasing our product. Please read these instructions carefully. Do not use the product until you have read and understood these instructions. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid damage to the product or injury to yourself.

NOTE: Inspect all parts for shipping damage. If any such damage is found, do not use the product. Contact the manufacturer for further information.

INTENDED USE

Toilet risers are designed for arthritis sufferers, people who have undergone hip surgery and others who have difficulty moving, standing and sitting. Raisers can increase the height of the toilet, making it easier to sit down and get up, especially for people with joint problems such as rheumatism or arthritis.

Reduced risk of falls: for people with balance problems, a stable booster seat can significantly reduce the risk of falling in the bathroom, one of the most accident-prone areas of the home.

CONTRAINDICATIONS

- Physical or mental limitations (e.g. visual impairment) that prevent safe handling.
- Hygiene problems: Although toilet risers are designed to improve hygiene, some are not sufficiently clean or are used by several people, which can lead to the spread of bacteria and infections. It's always important to ensure that the booster seat is properly cleaned and disinfected.
- Discomfort: some toilet risers can be uncomfortable for people with certain medical conditions, such as hemorrhoids or anal fissures. In these cases, hard materials may cause additional discomfort or pain.
- Reduced sensory perception: people with reduced sensory capacity in the lower part of the body should be wary of risers that are too hard or too cold, which can cause pressure sores or other skin lesions.
- Inability to sit up
- Severe imbalance

FEATURES

- Comfortable, ergonomic booster seats:
- Large support surface at front and back
- The contoured top surface of the booster seat is adapted to the shape of the body.
- Solid base for secure stabilization
- Soft thermoplastic rubber
- Side latches for toilet suspension
- Removable lid for optional use
- With bracket for secure, stable attachment to the toilet

MAXIMUM LOADS: 180KG

Size length 39.5cm width 36cm

Weight: 1.5 kg

MAXIMUM LOADS: 180KG

COMPONENTS:

Toilet raiser, instruction manual

NOTES:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person or
- (b) temporary or permanent deterioration of the health of the patient, user or other person or
- (c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

NOTE: it is forbidden to use the product other than for its intended purpose .

NOTES:

1. During use and installation and adjustment of mechanisms, there may be a risk of entrapment and/or compression of body parts of the user/guardian in the openings/gaps between components.
2. After making adjustments, stabilize the position by tightening the nuts/bolts.
3. Before each use of the product, check that all adjustable parts are securely fastened.
4. Use caution on wet or slippery surfaces.
5. Do not put your fingers into the seat openings that drain water. Doing so may cause crushing or pinching of fingers.
6. Do not use a damaged or broken toilet raiser.
7. Toilet raiser is not intended for permanent seating.

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

USE

- (a) stabilizing tab
- (b) mounting bracket
- (c) anti-slip protection
- (d) butterfly screw
- (e) front and rear notch
- (f) cover
- (g) lid hinge

First, install the handles together with the thumb screws in the outer mounting holes of the toilet raiser. The handles should be set wider than the width of the toilet bowl rim.

Then lift the toilet seat and place the toilet raiser directly on the toilet bowl, so that the mounting brackets are in the back of the bowl and the stabilizing tab rests against the outside of the front of the bowl. Then the thumb screws should be screwed in clockwise until a noticeable resistance is felt, so that the toilet raiser does not move on the toilet bowl

CLEANING AND CARE

Note: Before each use of the product, always check that all the

The adjustable parts are well fixed.

Clean with a damp cloth using neutral detergent.

Do not use high-pressure cleaners or aggressive, caustic cleaners for cleaning cleaning products.

MAINTENANCE OF THE PRODUCT:

- Easy to clean, free of external rough surfaces
- Do not use abrasive or scouring cleaners.
- If any defects or damage appear, discontinue use Product.

In order to ensure the long-term and safe operation of the product, it is recommended to regularly inspect all screws and mounting components. Before each use of the product, check that all screws are properly tightened and there are no signs of loosening.

Regularly check that the product is not missing any parts and that all components are in good working order. If you notice wear, mechanical damage or missing parts, immediately stop using the product and contact the manufacturer's service department to avoid the risk of an accident or further damage.

Watch for any unusual sounds or vibrations that may indicate loose components or other technical problems. Remember, when replacing screws or components, to use only original parts recommended by the manufacturer.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

METHOD OF DISPOSAL OF THE PRODUCT

Once the product has been taken out of service, the medical device can be disposed of as normal municipal waste. For its proper disposal, please contact your local authority or the nearest waste collection point to find out how to properly dispose of the product. By acting in accordance with local regulations, you are helping to protect the environment.

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NOTATKI

A series of horizontal dotted lines for taking notes, spanning the width of the page.

NOTATKI

A series of horizontal dotted lines for taking notes.

WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiańska 43 Tel. 22 518 36 00,
Fax 22 518 36 30 www.antar.net

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