AT52312
INSTRUKCJA UŻYTKOWANIA WÓZEK INWALIDZKI
USER MANUAL WHEELCHAIR
GEBRAUCHSANWEISUNG ROLLSTUHL
NÁVOD K POUŽITÍ INVALIDNÍ VOZÍK
NÁVOD NA POUŽITIE INVALIDNÝ VOZÍK
MODE D'EMPLOI FAUTEUIL ROULANT
GEBRUIKSAANWIJZING ROLSTOEL
INSTRUCCIONES DE USO SILLA DE RUEDAS
ISTRUZIONI PER L'USO SEDIA A ROTELLE
BRUKSANVISNING RULLSTOL

PL

WSTĘP

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ENG

INTRODUCTION

Before using your wheelchair, please read this user manual carefully. Incorrect use of the wheelchair may lead to health damage, injury or accidents. Thus, in order to use the wheelchair safely and successfully, please read this manual carefully.

SAFETY

The safest operation of the product is on a flat surface free of any obstacles. If possible, always use ramps when entering or leaving a building. Keeping your hands on the grip rims allows maximum balance.

Leaning

Do not lean over the side of the wheelchair as this can lead to loss of balance and falling.

Moving up hills

- Do not turn while moving on hills because of the risk of tipping over.
- The attendant should not push the wheelchair on hills diagonally.
- If the wheelchair user is going up a hill on his own, he should avoid moving on the hill diagonally.
- If it is necessary to go down a steep hill, the attendant should walk backwards with the wheelchair above him.
- If the user is going down a hill on his own, he should control the speed of the wheelchair at all times and not allow the product to overspeed. If necessary, another person should provide an assistance.

Kerbs

- If possible, always be assisted by a carer (if moving with a carer, the wheelchair must be provided with push handles).
- Avoid strong shocks when moving down off the kerb. Shocks can damage the product.
- Do not under any circumstances attempt to move onto high kerbs.
- Always exercise caution when passing kerbs or making turns on hills and never move backwards on a wheelchair without assistance from a carer.

Stairs

- Always have the assistance of two or more carers. Tilt the product slightly backwards for balance.
- Do not climb the stairs with the wheelchair.
- Do not use escalators. Use a lift.

General information

- Always check that BOTH brakes are blocked when getting in or out of the wheelchair.
- Do not stand on the footrests when getting on or off the product.
- It is not permitted to carry more than one passenger. Wheelchairs are designed for one user.
- Please take care that clothing does not block the wheels of the wheelchair.

Upholstery

- Use warm, soapy water to clean the upholstery.
- Upholstery cleaner/foam can also be used, but solvents should be avoided.
- Keep the wheelchair away from sources of fire that could cause injury to the user and damage the product.

NOTES:

In the event of a product-related 'serious incident' that directly or indirectly led to, could have led to or is likely to lead to any of the following:

- (a) the death of a patient, user or other person; or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

this 'serious incident' should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions or other distressing, unclear symptoms related to the use of the medical device, consult a healthcare professional.

ATTENTION: The manufacturer cannot be held responsible for damage caused by negligent maintenance, inadequate servicing or as a result of failing to observe the instructions in this manual.

ATTENTION: It is forbidden to use the product other than for its intended purpose!

ATTENTION: During use and handling of the product and when assembling and adjusting the mechanisms, there may be a danger that parts of the user's/companion's body become trapped and/or pinched in the openings/gaps between components. Perform these operations with particular care.

NOTE: There may be a risk of tipping over if the product is used incorrectly. Please follow the instructions for getting on/off/moving. Once the adjustment is complete, stabilise the position by tightening the nuts/bolts carefully.

Application/indications

The wheelchair is used for the individual transport of disabled people after spinal cord injury, with cerebral palsy, and with various lower limb injuries, neuromuscular diseases, after cerebral-cranial injuries or for geriatric patients.

Contraindications

Contraindications to sitting position. Conditions requiring stabilisation of the upper spine and head. Patients with psychiatric disorders should not be left unattended in a wheelchair.

GENERAL WARNINGS

- When getting on or off the wheelchair, make sure that both brakes are blocked.
- If the wheelchair is not being pushed or used make sure that both brakes are blocked.
- Avoid steep and uneven surfaces as this can cause the product to tip over.
- Avoid soft surfaces as the wheels may collapse.
- Move the wheelchair in a smooth manner in all conditions.
- Always observe your surroundings to identify hazards, such as furniture, doors, holes, hills, kerbs, entrances, etc.
- Always keep both feet on the footrests and ensure clothing is properly tucked and not blocking the wheels.
- Always move in a careful manner on hills.
- Always keep both hands on the push handles for proper control of the wheelchair (applies to the carer).
- Always remember to maintain balance.
- Check the frame and wheelchair components regularly and always follow the instructions in this manual during and after the warranty period.
- Keep the wheelchair away from sources of fire.
- Do not, under any circumstances, overload the product.
- Do not hang any heavy objects on the back of the wheelchair as this may affect the stability of the product, particularly when climbing hills.
- The wheelchair may only be used by one person.
- Do not use the wheelchair when the patient is unconscious.
- Do not use other means of transport to push the wheelchair.
- Do not lift the front wheels while the wheelchair is in motion, as this may result in a fall and serious injury or death.
- Do not get into the wheelchair while it is moving.
- Do not move the wheelchair too fast and do not make sudden turns.
- Do not move the wheelchair across a hill (in either direction) with a gradient of more than 7 degrees.
- Do not stand on the footrests when getting on or off the product.
- Do not climb stairs using a wheelchair, either alone or with a carer.
- Do not attempt to lift the wheelchair with moving (removable) parts. Lifting with moving parts of the product may lead to injury to the user and/or carer or damage to the product.
- Do not use the wheelchair if the backrest is folded. This changes the centre of gravity of the wheelchair and may cause the wheelchair to tip backwards. The backrest should only be folded when storing or transporting the wheelchair.
- No objects should be hung on the backrest-folding connector as this may cause the backrest to fold down and cause serious injury to the user.

COMPOSITION OF THE PRODUCT

The upholstery of the seat and backrest is made of black PVC leather, and the inner layer is fitted with a reinforcing fabric. The front wheels are made of 8-inch PVC wheels, with thickened iron front forks. The frame is made of steel.

WHEELS

- The front 8-inch wheels have a thickened metal fork, which is more durable.
- The rear wheel uses a robust 24-inch tyre that is shockproof, wear-resistant and has a long service life.
- The wheels are fixed.

Scope of delivery: Wheelchair, two footrests, calf stabilising belt, instructions for use.

USE

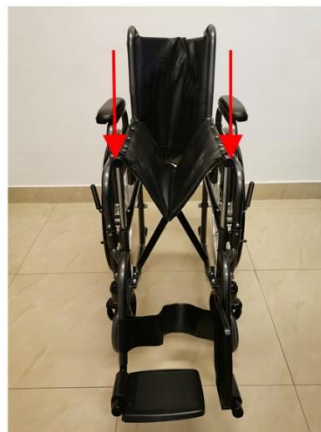
- After unpacking the package, check that it contains all parts according to the parts list.
- Ensure that all components have no visible mechanical damage. If in doubt, contact the manufacturer.
- Do not use the product until all its components have been properly assembled.

ASSEMBLING A WHEELCHAIR

- Fold the handles of the wheelchair outwards (figure 1)
- Press the seat frames together until the **wheelchair** is fully extended. (illustration 2 and 3)
- To attach the footrests, hang the footrest attachment handles on the footrest hangers and rotate them as far inwards as possible until the footrest locks into place and a distinctive click is heard
- Ensure that the footrests are locked.
- The footrest platforms should then be unfolded.
- The calf stabiliser strap should be fastened with Velcro on the footrest frames, threading it through the eyelets of the footrests.

1

2

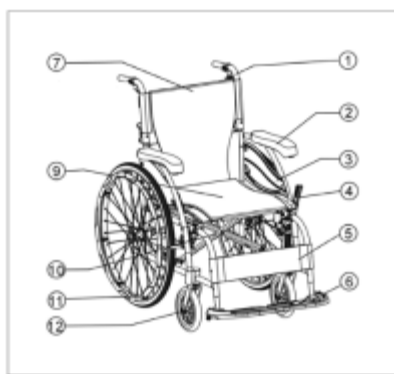
3

DISMANTLING OF THE WHEELCHAIR

All the steps described above should be carried out in reverse order.

CAUTION: When using and operating the wheelchair and when folding and adjusting the mechanisms, there may be a danger of trapping and/or pinching parts of the user's/companion's body in the openings/gaps between components. Perform these operations with particular care. Once the adjustment has been completed, stabilise the position by carefully tightening the nuts/bolts.

List of elements:

1. push handle
2. armrest
3. side panel
4. brake
5. calf support strap
6. footrest step
7. backrest upholstery
9. seat upholstery
10. cross frame
11. cast rear wheel
12. cast front wheel

MOBILITY IN A WHEELCHAIR

Grasp the grip rim around the outside of the large wheel at the front, then pull it towards you to make the wheelchair move. To release/brake, grasp the rim at the rear and hold gently. Caution, if the wheelchair moves quickly, injuries or burns to the hands (due to friction) may occur. It is advisable to use protective gloves when using the wheelchair intensively.

BRAKE MECHANISM

- To apply the brake, pull the brake lever towards the front wheel until the brake is tight against the rear wheel.
- Ensure that the brake lever does not bounce back on its own when attempting to move the wheelchair.
- To disengage the brake, pull the brake lever towards the rear wheel.
- To change the position of the brake mechanism, loosen the screw with a spanner.
- Move the whole mechanism towards the wheel or in the opposite direction and tighten the screw.
- When the brake is re-applied, the wheel should not turn (test the correct adjustment on a wheelchair with the patient)

PARAMETERS

Frame: steel
Maximum load: 115kg
Overall height: 90cm
Overall length: 103cm
Total width: 65cm
Seat height: 50cm
Seat width: 45cm
Seat depth: 40cm
Backrest height: 39cm
Weight: 16.7kg

This sign indicates the maximum weight of the user!

INSPECTION AND MAINTENANCE

Daily inspection

Before driving, check the following items. If any damage is found, contact your dealer for additional inspection of the product before use.

- Check brakes for wear or damage. Replacement or adjustment of the brakes should be carried out by a professional service.
- Check that the hand grips for the carer are not damaged and securely attached.
- Before using the wheelchair, check that all bolts and nuts are tightened. Check all parts for wear and replace if necessary. Check the adjustment of all parts. Check the brakes to make sure they work correctly.
- Wheels and tyres should be checked periodically for cracks and wear and replaced by a professional service if necessary.

CLEANING

To prolong the life of the wheelchair, it is recommended that it is cleaned regularly (preferably daily) especially if it is used in rainy or snowy weather. We recommend that the product is not used in rain or snow.

Wipe off with a damp cloth.

Upholstery

- Use warm, soapy water to clean the upholstery.
- Upholstery cleaning products/foams can also be used, but solvents and strong detergents should be avoided.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

TARGET PATIENT GROUP

The healthcare professional indicates on his own responsibility the use of the device for adults and children, taking into account the available variants/product's size/necessary functions/available sizes and indications, bearing in mind the information provided by the manufacturer.

HOW TO DISPOSE OF THE PRODUCT

The medical device can be disposed of as normal municipal waste.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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