AT52304

**WÓZEK INWALIDZKI, ELEKTRYCZNY INSTRUKCJA OBSŁUGI
ELECTRIC WHEELCHAIR USER MANUAL
ELEKTOROLLSTUHL GEBRAUCHSANWEISUNG
UŽIVATELSKÁ PŘÍRUČKA K ELEKTRICKÉMU INVALIDNÍMU VOZÍKU
NÁVOD NA POUŽITIE ELEKTRICKÝ INVALIDNÝ VOZÍK
MANUEL D'UTILISATION DU FAUTEUIL ROULANT ÉLECTRIQUE
GEBRUIKERSHANDLEIDING ELEKTRISCHE ROLSTOEL
MANUAL DE USUARIO DE LA SILLA DE RUEDAS ELÉCTRICA
MANUALE D'USO DELLA SEDIA A ROTELLE ELETTRICA
ANVÄNDARHANDBOK FÖR ELEKTRISK RULLSTOL**

MD

PL

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En

INTRODUCTION

Before using your power wheelchair, read and follow the information, warnings, and cautions in this manual. If this manual contains information that you do not understand, or if you need further assistance with assembly or operation, please contact your dealer. Following the instructions, warnings, and cautions in this manual increases the safety of using the product. The manufacturer shall not be liable for any damage and/or injury resulting from improper use of the device or use not in accordance with the instructions and warnings contained herein.

APPLICATION

The product is designed for people who cannot or do not wish to move independently in a wheelchair using the strength of their own hands, and for those who wish to actively participate in everyday life.

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling of the product.

CAUTION! Failure to observe the warnings in this manual may result in personal injury.

ATTENTION! Failure to follow the instructions in this manual may result in damage to the power wheelchair.

ATTENTION! It is forbidden to use the product for purposes other than its intended use.

DESIGN AND OPERATION

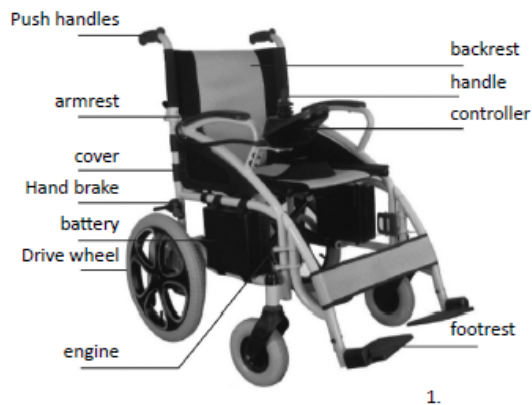
The power wheelchair consists of the following main components (Figure 1):

Seat frame: equipped with a mechanism that allows the seat to be folded along its axis, enabling convenient storage or transport.

Control system: consisting of a controller, a battery compartment, two motors, a set of drive mechanisms, and drive wheels.

The battery compartment is located under the seat. The wheelchair is equipped with a 24V, 12Ah battery with overload protection. The battery charger socket is located on the rear side of the controller.

The drive mechanism is equipped with two manual levers. These levers allow the wheelchair to be stopped.

TECHNICAL DATA

Overall size (Length × Width × Height): 1030 × 600 × 915 mm

Seat height: 540 mm

Seat width: 440 mm

Seat depth: 470 mm

Armrest height: 200 mm

Backrest height: 440 mm

Battery weight: 2.9 kg

Weight of wheelchair with battery: 36.5 kg

Maximum speed: 6 km/h

Braking distance at 6 km/h: ≤ 1500 mm

Maximum turning radius: ≤ 1200 mm

Maximum load: 120 kg

Approximate range on a single charge: 20 km

Static stability: ≤ 9°

Dynamic stability: ≤ 6°

Maximum climbing angle: 6°

Motor specification: 24V / 250W × 2

Battery specification: 24V / 12Ah × 1

Maximum controller supply current: 50 A

Battery charging current: 2 A

Front wheel: solid tyres (no inflation required), outside diameter 210 mm

Rear wheel: pneumatic tyres, outside diameter 406 mm

This symbol indicates the maximum user weight.

The stated dimensions may vary within a range of ±2 cm.

NOTE It is forbidden to stand on the footrests, as this may lead to a dangerous fall and injury.

ASSEMBLY

Remove the wheelchair from the packaging and place it on the ground. Pull the chair frame apart in opposite directions and press the two tubes on either side of the seat base down into the grooves on either side of the frame (Fig. 2).

Pull the handle upwards to lock the wheelchair backrest and prevent it from folding (Fig. 3).

Connect the battery cable plug to the socket located in the frame under the seat (Fig. 4).

Remove the screws from the controller base, mount the controller to the bracket on the armrest, and tighten the screws (Fig. 5).

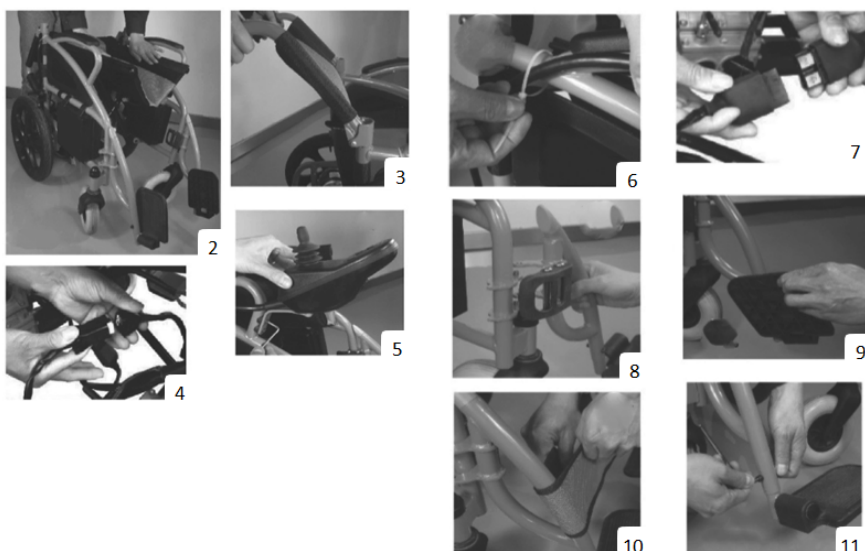
Connect the controller wires marked R and L to the right and left motors respectively.

Secure the wiring harness to the frame with plastic ties (Fig. 6).

Fit the footrests to the left and right sides of the frame (Fig. 8), then rotate and unfold them (Fig. 9).

Place the calf support strap over the right and left footrest frames and fasten it securely.

Adjust the height of the footrests: remove the lock nuts on both sides, remove the bolts (Fig. 11), set the desired height, then reinsert the

CONTROLLER (METHOD OF USE)

The controller is a key component of the wheelchair and contains all controls required for operation (Fig. 12).

The controller consists of:

1. Joystick
2. Battery level indicator
3. On/Off button
4. Speed indicator LEDs
5. Acceleration button
6. Mute button
7. Audible Signal button
8. SOS button

The controller is usually mounted on one of the armrests and is powered by the battery used to power the motors.

On/Off button

The On/Off button controls power to the controller electronics and the motor. Do not use the On/Off button to stop the wheelchair except in emergencies. Using it for this purpose may reduce the service life of the product.

Joystick

The joystick controls the speed and direction of travel. Speed is controlled by the degree of joystick deflection from the centre position.

When the joystick is released, it returns to the centre position and the brake is applied automatically.

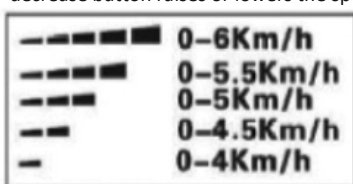
NOTE: If the wheelchair moves unintentionally, immediately release the joystick to stop the device.

Horn button

Press the button to activate an audible warning signal.

Acceleration button

When the power is switched on, the speed indicator LEDs show the current maximum speed setting. Each press of the speed increase or decrease button raises or lowers the speed setting shown by the LEDs.

Move the brake lever backwards to lock the wheels and prevent accidental movement. To drive, move the brake lever forwards to unlock the wheels (Fig. 13).

NOTE: Use the brake only when necessary (e.g. on inclined surfaces). Incorrect use may cause loss of control and may pose a risk to the user.

SAFETY BELT

For safety, always fasten the safety belt before using the wheelchair. Engage the belt buckle and ensure it clicks into place (Fig. 14).

BATTERY CHARGING

The wheelchair is supplied with an external charger, which allows the battery to be charged quickly and safely.

NOTE: Charge the battery only using the charger supplied with the wheelchair. Do not use automotive battery chargers. Before first use, fully charge the battery (do not exceed 8 hours).

Charging procedure:

Ensure the controller is switched off and the wheelchair wheels are locked.

Connect the charger plug to the charging socket on the controller (Fig. 15).

Plug the charger into a wall socket.

Charging starts when the red LED on the charger lights up. The green LED indicates a full charge.

The battery level is indicated by 5 LEDs (regardless of colour).

A full charging cycle takes 8–12 hours.

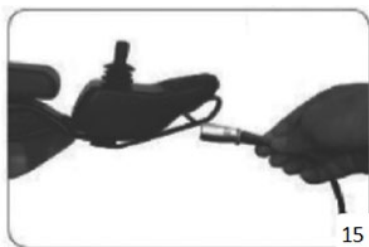
When charging is complete, unplug the charger from the wall socket first, then disconnect it from the wheelchair. Store the charger in the bag located at the rear of the backrest.

Separate battery charging:

The battery can be charged separately: connect the charger plug to the socket on the battery box (Fig. 15-1).

The battery box has a one-button quick-release mechanism. Press the button and lift the battery box to remove it for external charging (Fig. 15-2). Follow the above steps in reverse order to install the battery box.

In the event of overloading the motors, there is an overload protection system that cuts off the power supply to protect the motors and the electrical installation. Restoring operation once the protection system has been activated requires the intervention of a specialist - remove the screws located on the battery compartment, replace the fuse and then reapply the battery compartment cover.

SAFETY BELT

To ensure your safety, remember to wear your seatbelt every time you use the wheelchair.

PRECAUTIONS

Ensure that the controller is correctly installed and that the handlebar is in the central position, pointing vertically upwards. The user should sit in the wheelchair, fasten the seat belt and rest the back against the backrest.

When the power is switched on, first check that the maximum speed indicated by the controller lamp is suitable for your needs. If it is necessary to change the maximum speed, adjust it using the up or down speed button on the control panel. For safety reasons, it is recommended to start adjusting the maximum speed from lower values. In the event that the wheelchair starts to move accidentally, immediately release the handle, which will stop the engine.

Note: The user's feet must be on the footrests.

Note: Keep your forearms on the armrests when using the wheelchair.

Note: Make sure the wheels are locked before using or putting away the product.

DANGER

The use of the wheelchair is prohibited:

1. When expressly forbidden in this manual, i.e. driving on steep gradients, overcoming excessively high off-road obstacles, driving on motor vehicle traffic routes, etc.
2. In certain locations or on surfaces that present hazards (e.g. excessive ice, wet surfaces).
3. If a fault is found in the control system or other key components.

ELECTROMAGNETIC INTERFERENCE

The wheelchair may be used in areas affected by electromagnetic interference generated by radio transmitters such as radios, wireless intercoms, mobile phones, radars, etc. Electromagnetic interference may, however, affect the operation of the wheelchair.

NOTE: Electromagnetic interference (EMI) may affect the operation of the device. If the device is found to malfunction due to electromagnetic interference, contact the service centre immediately. The manufacturer shall not be held liable for any damage resulting from failure to comply with the above recommendation.

FOLDING AND STORING THE WHEELCHAIR

Store the wheelchair in a dry, cool place away from direct sunlight.

For transport or storage, fold the wheelchair as follows:

Place the wheelchair on the ground. Switch off the power supply. Raise the footrests and rotate them by 90° (Fig. 17).

Unfold the leg rest section, rotate it by 90°, and pull it out completely (Fig. 18).

Squeeze the wheelchair lock release levers located on both sides of the push handles and rotate them (Fig. 19).

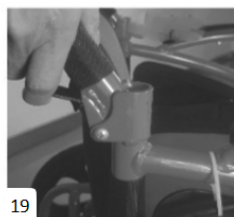
Pull the seat base upwards from the centre and fold the wheelchair (Fig. 20).

17

18

19

20

NEW FEATURES OF AT52304 II GEN

Asymmetrically positioned motors to reduce the folded dimensions (Fig. 21).

One-button quick-release battery box and separate charging port (Figs. 22, 23).

New effective front shock-absorbing spring system (Fig. 23).

Detachable anti-tipping casters (Fig. 25).

21

22

23

24

25

SCOPE OF SUPPLY

Wheelchair, controller, 2 footrests, calf stabilisation strap, instruction manual.

WARRANTY

1.1 Serial number

For after-sales service and warranty purposes, the vehicle identification number must be entered.

MODEL: AT52304 II GEN

SERIAL NUMBER:

1.2 Warranty terms

1.2.1 Repairs will be carried out free of charge if the issue is due to manufacturing defects or faulty parts during the warranty period (hereinafter referred to as "warranty repairs").

1.2.2 Warranty repairs may only be carried out by ANTAR service.

1.2.3 ANTAR is not responsible for transport costs related to repair or replacement.

1.2.4 Replaced parts remain the property of ANTAR.

1.3 Warranty period and parts covered

Electrical parts (motor, controller, charger): 12 months from the date of purchase

Wheelchair frame: 12 months from the date of purchase

Batteries: 6 months from the date of purchase

1.4 Parts not covered by warranty

The items listed below are not covered by the warranty:

1.4.1 Wear and tear resulting from use and passage of time (e.g. normal ageing of materials, deterioration of paint or plastics, etc.).

1.4.2 Damage caused by lack of maintenance or improper handling and storage.

1.5 The warranty does not cover issues resulting from:

1.5.1 Improper storage.

1.5.2 Modification or installation of non-original components.

1.5.3 Damage caused by using parts other than original ANTAR parts or parts not recommended by ANTAR.

1.5.4 Excessive or improper use, such as driving over kerbs or overloading the wheelchair.

1.5.5 Use in areas where the wheelchair should not normally be operated.

1.5.6 Damage or corrosion caused by environmental factors such as chemical exposure, tree sap, stones, hail, strong winds, flooding, etc.

1.5.7 Any unauthorised modifications to materials or components.

1.5.8 The wheelchair must not be pulled or pushed by another vehicle while occupied, or when the rear wheels are not set to “freewheel” mode.

CARE AND MAINTENANCE

1. Care and maintenance after use

Switch off the power supply (it is recommended to disconnect all electrical connections). The wheelchair should not be used by children or persons with cognitive impairments. Store the wheelchair at room temperature to reduce potential deformation caused by temperature changes and to ensure efficient operation over a long period. Clean the wheelchair with a soft cloth and mild detergent, and dry it after cleaning. Do not use abrasive objects or strong detergents.

2. Checking correct operation

Several checks should be carried out before each ride to ensure proper function. In addition, periodic inspections are recommended: weekly, monthly, and every 6 months, as shown in Table 2.

3. Troubleshooting

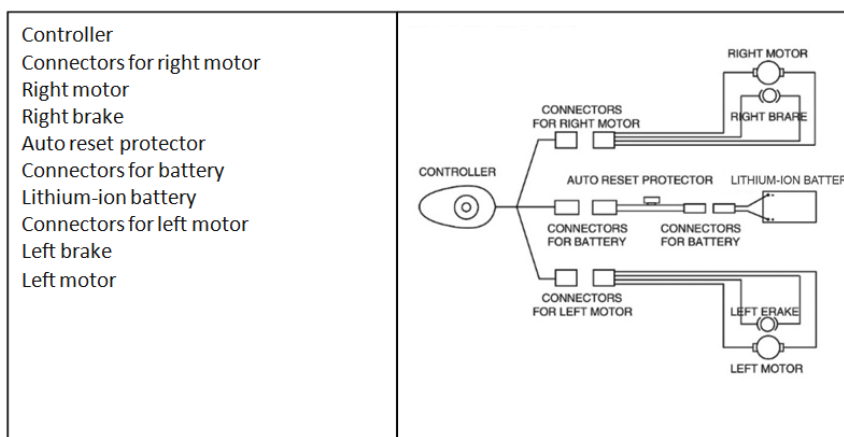
The wheelchair is equipped with an automatic fault notification system. Malfunctions are indicated by LEDs on the controller and an audible signal. Fault codes should be interpreted according to Table 3. If the fault cannot be resolved and the alarm persists, contact an authorised service representative.

Table 2 Error messages

ERROR NUMBER	LEDS	DIAGNOZE
1	1 LED	Low battery charge
2	2 LEDS	Left engine failure
3	3 LEDS	Failure of the left brake
4	4 LEDS	Failure of the right engine
5	5 LEDS	Failure of the right brake
6	6 LEDS	Controller overload
7	7 LEDS	Failure of the control lever
8	8 LEDS	Controller system failure
9	9 LEDS	Failure of the control system

Table 3 Unit checks

Controlled element	Daily	Once a week	Every month	Every six months
Each part			X	
Turning, guiding, driving, setting, folding, etc.		X		
Braking	X			
Connecting cables		X		
Batteries and charging	X			
Front wheels		X		
Rear wheels		X		
Tyre wear			X	
Tyre damage	X			
Degree of wear of push handles and backrest and seat	X			
Engines				X
Controller		X		
Cleanliness	X			

ELECTRICAL DIAGRAM

NOTE: For intensive use, it is recommended to replace the batteries every 6 months. The warranty period for the batteries is 6 months.

NOTE: Check the rear tyre pressure (2.8–3 bar) before first use. The pressure should also be checked periodically.

SERIOUS INCIDENTS (MDR)

In the event of a product-related “serious incident” that directly or indirectly led to, could have led to, or may lead to any of the following:

- (a) death of a patient, user, or other person; or
- (b) temporary or permanent deterioration of the health of a patient, user, or other person; or
- (c) a serious threat to public health

the “serious incident” must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. NOTE: If you experience pain, allergic reactions, or other worrying symptoms related to the use of the medical device, consult a healthcare professional.

WARNING: During use and handling of the product and when assembling or adjusting the mechanisms, there is a risk of trapping and/or pinching parts of the user’s or assistant’s body in openings or gaps between components. Perform these operations with particular care.

NOTE: There is a risk of tipping over if the product is used incorrectly. Please follow the instructions for getting on/off and moving. After adjustments, stabilise the position by tightening nuts/bolts securely.

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

HOW TO DISPOSE OF THE PRODUCT

Dispose of the electrical medical device in accordance with the obligatory rules. Handle in a manner appropriate to the disposal of electrical and electronic equipment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR MEDICAL Sp. z o.o. ,03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.
10. The warranty does not cover parts and components subject to natural wear and tear, such as wheels, tires, rubber pads, fuses, etc., the wear of which results from normal use of the product. The warranty period for batteries is 6 months.

ANTAR MEDICAL Sp. z o.o.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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