AT52339

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En

Thank you for purchasing our product. Please read the user manual carefully. Do not use the product before reading and understanding these instructions. If you do not understand the warnings, notes, and recommendations, contact a healthcare professional or the seller to avoid product damage or personal injury. **WARNING:** All parts must be checked for damage. Before first use, thoroughly inspect the technical condition of the product and ensure that all parts are complete and properly assembled (if applicable, pay particular attention to screws, nuts, rivets and other fastening and structural components). Make sure that the product is not defective or damaged. If any damage or missing parts are found, DO NOT use the product. For further information, contact the seller.

TARGET PATIENT GROUP: Individuals suffering from conditions, dysfunctions, or injuries for which this product is intended for treatment/rehabilitation or compensation (see the section on intended use in this manual). The product may be purchased independently by the user or based on the recommendation of a physician, therapist, or other specialist. In both cases, the available sizes/required functions and product variants, indications and contraindications for use, as well as the information provided by the manufacturer must be taken into account.

INTENDED USE

An electric wheelchair is a vehicle designed for the mobility of a person with disabilities. It is powered by a motor, the design of which limits the driving speed to walking pace.

The AT52339 mobility scooter has been designed for users with a maximum weight of up to 120 kg, who move in a seated position. Safe use of the device requires an appropriate level of motor skills, vision, and cognitive ability; therefore, consultation with a physician before use is recommended.

The AT52339 model is intended for both indoor and outdoor use, within a wide temperature range from -20°C to +40°C. It can be used during light rain; however, prolonged use in heavy rain or snow is not recommended.

Thanks to relatively large wheels, the scooter allows driving on unpaved surfaces, moderately uneven terrain, and overcoming obstacles up to 4 cm in height. The maximum safe incline is 9° (16%) and must not be exceeded.

The vehicle is easy to operate, and the braking system ensures smooth and quick stopping after releasing the acceleration lever. Under standard conditions, the scooter range is approximately 16 km; however, this may decrease when driving uphill, on uneven terrain, or when frequently overcoming obstacles.

Before first use, carefully read the user manual and perform a visual inspection of the device to ensure that it has no visible damage. In case of any doubts, contact the seller.

GENERAL SAFETY RULES

MANDATORY! Before first use, carefully read this manual and make sure you understand all the rules for using the AT52339 mobility scooter.

The AT52339 mobility scooter is a medical device intended to improve user mobility. Its safe use requires caution, concentration, and adaptation of driving style to environmental conditions.

Drive slowly until you become familiar with all functions and features of the scooter.

In crowded areas, move at reduced speed.

Exercise particular caution with pedestrians and obstacles.

Use the scooter only in accordance with local regulations.

PROHIBITED! While driving, keep your hands away from tires and wheels. Remember that loose clothing may be caught by the tires and wheels.

WARNINGS REGARDING USE

WARNING! Do not exceed the maximum permissible load of 120 kg (user + luggage).

WARNING! The scooter is intended for one person only. Do not carry passengers, including children.

WARNING! Do not use the scooter on stairs or escalators.

WARNING! Do not overcome obstacles higher than specified and always approach them frontally.

WARNING! Do not use the scooter in heavy rain, high humidity, or extreme temperatures.

WARNING! Scooter surfaces may become hot – exercise caution.

DRIVING SAFETY

Always adjust your speed to road conditions. Pay particular attention to pedestrians and adapt your driving accordingly. In crowded places, always move at half speed.

Reduce speed before turning – turning too quickly may cause the vehicle to tip over.

Maintain a stable body position while driving.

Avoid sudden maneuvers and abrupt braking.

WARNING! Do not lean excessively forward or sideways – this may result in loss of balance and tipping of the scooter.

PROHIBITED! Do not place hands or feet near moving parts (wheels, mechanisms).

SAFETY ON SLOPES AND RAMPS

Do not exceed the permissible incline.

Do not travel across or diagonally on slopes.

When descending, set the minimum speed.

When ascending, maintain a constant speed.

WARNING! Never use freewheel mode on a slope.

PRE-USE INSPECTION

Before each ride, check: battery condition, brake operation, electrical connections, tire condition, and proper functioning of all features. If any irregularities are detected, do not use the scooter and contact service.

BATTERIES AND ELECTRICAL SAFETY

WARNING! Do not allow the battery to come into contact with metal objects – risk of short circuit and electric shock.

WARNING! Use only recommended batteries and chargers.

WARNING! Do not charge the battery near flammable materials.

WARNING! Do not allow children near the scooter during charging.

CHILDREN AND BYSTANDERS

WARNING! Do not allow children to operate the AT52339 mobility scooter.

WARNING! Do not leave the scooter unattended in the presence of children.

WARNING! Do not carry passengers. This may cause loss of stability.

MODIFICATIONS AND PARTS

WARNING! Do not modify the scooter or use non-original parts and accessories.

WARNING! Do not lift the scooter by detachable components (e.g., armrests, seat).

EMERGENCY SITUATIONS

WARNING! If the scooter does not stop properly, immediately use the brake, switch off the power, and contact service.

WARNING! After an accident or damage, the scooter must be checked by qualified service before reuse.

ELECTROMAGNETIC INTERFERENCE (EMI/RFI)

WARNING! Devices such as mobile phones or radios may cause unintended scooter movement.

WARNING! Do not use radio devices while driving and avoid staying near strong signal sources.

TRANSPORT IN A MOTOR VEHICLE

During transport, the AT52339 electric mobility scooter must be transported without the user, in the trunk or cargo space of the vehicle, with batteries removed and properly secured against movement.

Detachable elements should be removed or firmly secured.

WARNING! Do not sit on the scooter during transport.

WARNING! The scooter and batteries must be properly secured.

WARNING! Do not transport batteries with flammable materials.

INDICATIONS

The product is intended for use in cases of:

difficulty walking independently or rapid fatigue during movement,
musculoskeletal disorders (e.g., joint diseases, injuries, degeneration),
reduced muscle strength or balance disorders,
conditions after injuries or surgeries requiring relief of the lower limbs,
neurological diseases affecting walking ability,
advanced age associated with limited mobility.

The scooter is intended for use by one person who is able to operate the device independently or use it under caregiver supervision.

CONTRAINDICATIONS: Physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product.

Lack of ability to safely operate the vehicle in outdoor environments (e.g. lack of orientation in pedestrian or road traffic).

Uncontrolled body movements (e.g. tremors, spasticity) that may hinder operation.

Episodes of loss of consciousness, dizziness, or other conditions that may occur suddenly أثناء driving.

Inability to maintain a stable seated position.

Significant limitations in reaction time and motor coordination.

Use without supervision in the case of persons requiring constant care.

LIST OF COMPONENTS: Electric wheelchair, battery, charger, cushion, basket, user manual, tool kit.

TECHNICAL SPECIFICATION

Dimensions when unfolded (length x width x height): 970 × 530 × 900 mm

Dimensions when folded (length x width x height): 970 × 530 × 290 mm

Seat dimensions (W × L × T): 390 × 340 × 30 mm

Motor: 24V 250W (2 × brushless)

Battery: 24V 8A lithium, removable

Battery weight: 2 kg

Charger: 24V 2A

Controller: 24V 60A brushless

Steering column: height adjustment

Speed: adjustable 0–6 km/h

Brake: electromagnetic ABS

Braking distance: ≤ 1000 mm

Minimum turning radius: ≤ 1500 mm

Ground clearance: 80 mm

Obstacle climbing ability: 40 mm

Backrest height: 830 mm

Armrest height: 720 mm

Distance between armrests: 435 mm

Static stability: ≥ 9°, < 15°

Dynamic stability: ≥ 6°, < 10°

Lighting: front and rear light

Basket: front basket

Basket dimensions: 310 × 230 × 210 mm

Maximum basket load: 25 kg

Range: 15 km

Front wheel: 9" pneumatic

Rear wheel: 9" pneumatic

Maximum load: 120 kg

Net weight: 18.6 kg

Gross weight: 28 kg

MATERIAL COMPOSITION

Frame: aluminum alloy

Motor: metal

Battery: lithium battery

Battery housing: plastic

Charger: plastic

Armrests: PU foam

Seat: PU foam

Backrest: PU foam

Mudguards: plastic

Front basket: nylon fabric

Rear basket: none

Front wheel: rubber, aluminum alloy rim

Rear wheel: rubber, aluminum alloy rim

Steering system: ABS

Plastic components: ABS

NOTE! INDIVIDUAL PRODUCT DIMENSIONS MAY VARY WITHIN A TOLERANCE OF ± 2 CM

This symbol indicates the maximum user weight

1. Control panel

The control panel includes the display, speed adjustment knobs, key, lights, and horn.

2. Charging port

For charging.

3. USB socket

For mobile phone, etc., output voltage 5 V.

4. Hook

Used for hanging items, maximum load 3 kg.

5. Steering column adjustment knob

Used to adjust the angle of the steering column.

6. Front lights

7. Removable battery

The battery can be removed.

8. Basket / storage compartment

Maximum load 6 kg.

9. Armrest adjustment knob

After loosening the screws, the armrest can be adjusted outward. It should then be tightened again.

10. Rear armrest folding button

Press the button to fold the armrest backward.

11. Seat cushion

12. Armrest

The seat width can be adjusted, and the backrest can be reclined backward.

13.Backrest

Can be folded forward.

14.Folding handle

Special care must be taken when using the folding handle. The seat folding mechanism is locked by the handle. Before use, press the seat with your hand to make sure that it is properly locked. The condition of the folding handle should be checked regularly. More information can be found below.

15.Rear light

16.Rear bumper

17.Anti-tip wheel

18.Mudguard

1.Power switch

When the power is switched off, the scooter cannot move electrically.

Turn the key clockwise to switch on the power and enable driving.

2.Battery display

After switching on the power, the battery level indicator shows its status.

The green area indicates a high charge level, while the red area indicates a low level and the need for immediate charging.

3.Speed control

The speed control allows the maximum forward driving speed to be set in the range from 0 to 6 km/h.

Turning the knob fully to the left sets the lowest speed, and fully to the right sets the highest speed.

4.Front and rear light switch

Turn on the lights when driving at night or in conditions of limited visibility.

5.Horn button

Pressing the button emits an audible signal.

To prevent accidents and ensure safe driving, use it when necessary.

6.Forward and reverse drive lever


Push the right lever forward to drive forward; push the left lever forward to drive backward.

When the lever is released, it automatically returns to the "middle" position, which activates the electromagnetic brake and stops the scooter.

7.Steering handlebar

While driving, hold both hands firmly on the handlebar.

OPERATING PRINCIPLE OF THE FOLDING LEVER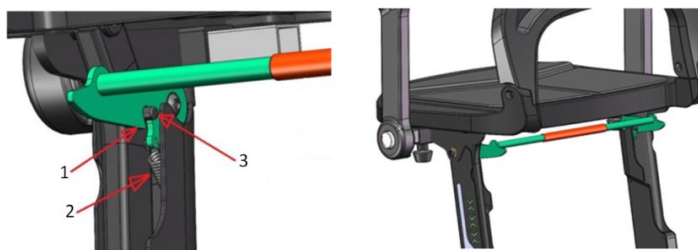


1.Folding handle

2.Spring

3. Pin/bolt

BRAKE CONTROL LEVER

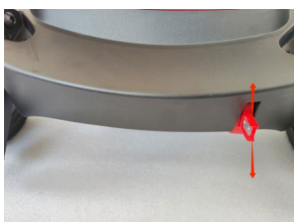POWER SYSTEM

Battery charging procedure and indication

The charger should be connected to the charging port located on the battery housing. The charger has protection against incorrect connection (it cannot be inserted incorrectly).


During charging, a red indicator light is on, indicating the charging process. Once the battery is fully charged, a green indicator light comes on, indicating the end of charging.

WARNING! Never connect the charger incorrectly, as this may damage the charging port.

Overload protection and fuse


The battery housing contains overload protection. This is an important safety feature that protects the battery from damage by controlling excessive current flow during charging.

The fuse is integrated with the overload protection. In the event of a short circuit, the fuse blows, protecting the entire electrical system.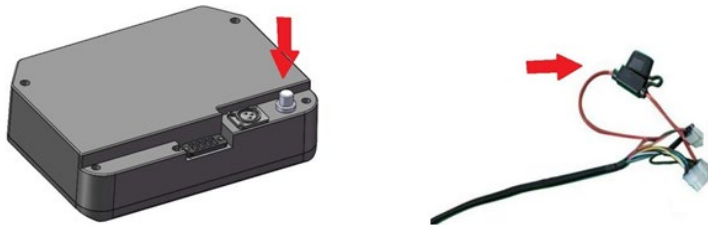

MANUAL AND AUTOMATIC SWITCHING OF THE BRAKE LEVER

The brake lever allows switching between “electric drive” mode and “manual pushing” mode.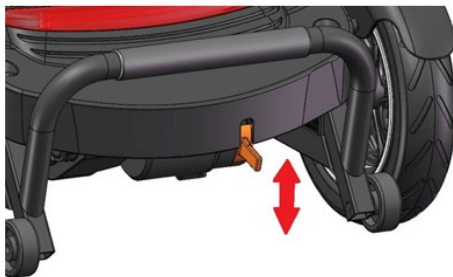

Electric drive: set the lever to position D (locked position) – the vehicle moves using the motor.

Manual mode: set the lever to position N (unlocked position) – allows the vehicle to be moved manually.

If the vehicle does not move and only an audible signal is heard, check whether the brake lever is in position D.

WARNING! Do not leave the brake lever in the unlocked (N) position while driving downhill – the brake will not function.

Note: During use, use only the power supply from the left battery.

BATTERY INSTALLATION AND REMOVAL

Removal: Grip the battery handle. Due to the hook-and-loop fasteners on the bottom of the battery housing, gently move the battery forward and backward and side to side before lifting it up.


Installation: Position the battery housing correctly relative to the socket at the bottom and slide it into place.

WARNING! During transport, always make sure that the AT52339 mobility scooter and its battery are properly secured. The battery must be kept in an upright position and must not be transported together with flammable materials.

Note: To prevent power disconnection while driving (e.g. on uneven surfaces), the battery is mounted in a special way. After installation, it must be pressed down firmly so that the hook-and-loop fasteners on the bottom adhere properly.

Note: Due to the hook-and-loop fasteners located on the bottom of the housing, greater force is required for removal and the battery must be pulled upward.

Note: Make sure that the battery cables are connected correctly.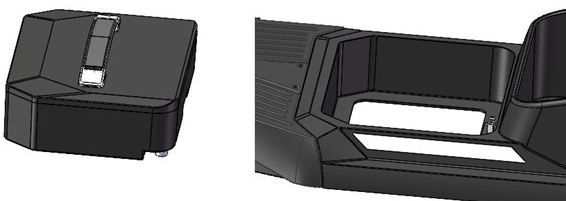

FOLDING, UNFOLDING, AND TRANSPORT OF THE SCOOTER

The scooter with removable battery consists of four main parts: frame, battery, seat, and basket. During disassembly, follow the instructions and exercise caution.

Folding procedure:

Folding the scooter

With one hand, gently move the steering column while turning the knob at the same time – this will make it easier to tighten it.

With one hand, lift the rear part of the seat, grasp its center, and lift it upward.


While lifting the seat, the folding lever will automatically be locked by the spring on the pin – you will hear a “click.”

Press the seat to make sure that it is properly locked.

Fold the armrest forward until resistance is felt.

WARNING! The folding lever must be properly locked by the pin. Otherwise, there is a risk of falling while seated.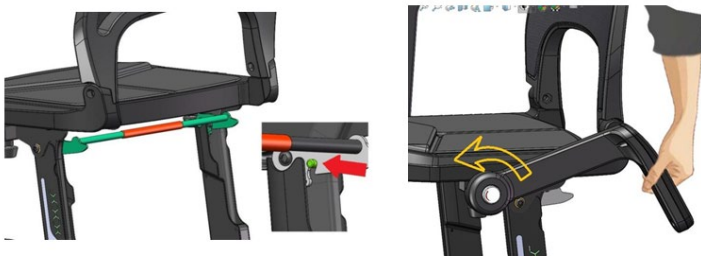


Unfolding procedure:

Perform the steps in reverse order.

With one hand, lift the folding lever, and with the other hand press the seat downward.

With one hand, press the button, and with the other hand rotate the armrest backward.

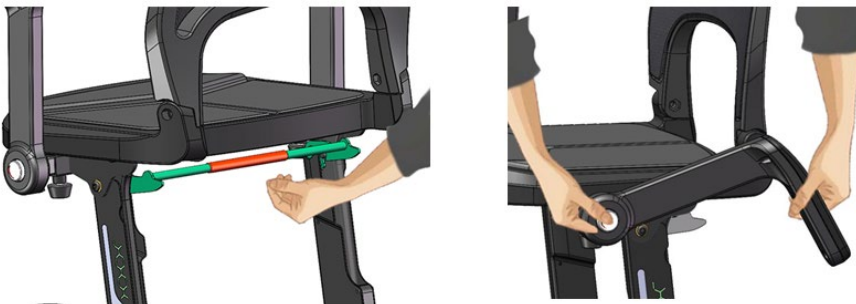Transport and carrying: When carrying the scooter, hold it by the front and rear protective bars (bumpers), as shown below.

CHARGING THE BATTERIES

Insert the charger plug (8) into the charging socket (10) located on the housing.


Connect the charger power cord (9) to the mains socket.

After charging is completed, disconnect the power cord (9) from the socket, and then remove the charger plug (8) from socket (10).

Under ideal storage conditions, fully charged and unused batteries should be recharged every 20 days.

If you plan a longer break from using the AT52339 mobility scooter, it is recommended to charge it for two days and then disconnect the batteries.

If the scooter has not been used for a long time, the batteries should be charged for at least 24 hours before driving.

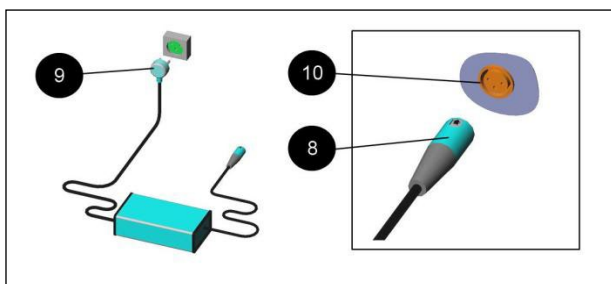Recommended charging cycle

Use the scooter during the day as needed or until the battery level is low (according to the LCD indicator).

At the end of the day, charge the batteries overnight.

There is no need to disconnect the batteries after charging is complete – the charger is automatic and stops charging by itself.

There is no need to charge after every ride unless you do not plan to use the scooter again that day.

As long as the charger plug is in the socket, the electronic system of the AT52339 mobility scooter automatically cuts off the power supply – driving is impossible.

The supplied charger is intended for lithium batteries. Use only the recommended type of charger. If a different one is used, consult the seller.

The battery is equipped with a protection circuit. After it is activated, the battery must be recharged to restore its operation.

NOTE: In the event of a “serious incident” related to the device that directly or indirectly led, could have led, or may lead to any of the following:

- a) death of a patient, user, or other person, or
- b) temporary or permanent deterioration of the health of a patient, user, or other person, or
- c) a serious public health threat

such a “serious incident” must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the United Kingdom, the competent authority is the Medicines and Healthcare products Regulatory Agency (MHRA).

NOTE: In the event of pain, allergic reactions, or other disturbing or unclear symptoms related to the use of the medical device, consult a healthcare professional. **NOTE:** It is prohibited to use the product in any manner other than its intended purpose.

CLEANING, MAINTENANCE AND STORAGE

Cleaning

The AT52339 mobility scooter should be cleaned as needed using a soft, damp cloth and mild detergents. After cleaning, wipe the surface with clean water and then dry thoroughly.

Do not use pressure washers or large amounts of water, as this may damage electrical components. Cleaning should be carried out at temperatures above 0°C.

Maintenance

The scooter requires regular inspections despite low maintenance requirements:

Daily: check the battery charge level, horn operation, and cleanliness of the battery compartment.

Weekly: check the operation of the drive lever, brake, and the stability of wheels and connections.

Monthly: check tire wear and proper driving and braking performance.

Periodically: inspect the condition of cables, connections, and all structural components.

All repairs, adjustments, and battery replacement should be carried out only by qualified service personnel.

Storage

In case of prolonged non-use (e.g., over 4 months), the scooter should be stored in a dry, clean, and secure place, preferably with access to power supply. It is recommended to cover the scooter during storage. The battery should be recharged at least every 20 days. Do not store the scooter at temperatures below 0°C. Low temperatures (below 10°C) may reduce battery performance and capacity. **WARNING!** Do not allow the battery to freeze – this may cause permanent damage.

DISPOSAL: At the end of its service life, the scooter must be disposed of in accordance with applicable local, national, and EU regulations regarding waste electrical and electronic equipment (WEEE Directive). The device must not be disposed of with unsorted municipal waste. The scooter contains recyclable components such as plastics, electronic components, and batteries. Particular attention should be paid to the battery, which contains substances requiring appropriate handling (including lithium compounds) and should be delivered to specialized collection points. For information on proper disposal methods, contact local authorities or an authorized seller. **NOTE:** Improper disposal may pose a risk to the environment and human health.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR MEDICAL Sp. z o.o. ,03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.
10. The warranty does not cover parts and components subject to natural wear and tear, such as wheels, tires, rubber pads, fuses, etc., the wear of which results from normal use of the product. The warranty period for batteries is 6 months.

ANTAR MEDICAL Sp. z o.o.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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